# Information on the Performance of Global Endometrial Ablation Devices: A Summary of Safety and Effectiveness Data from the FDA<sup>3</sup>

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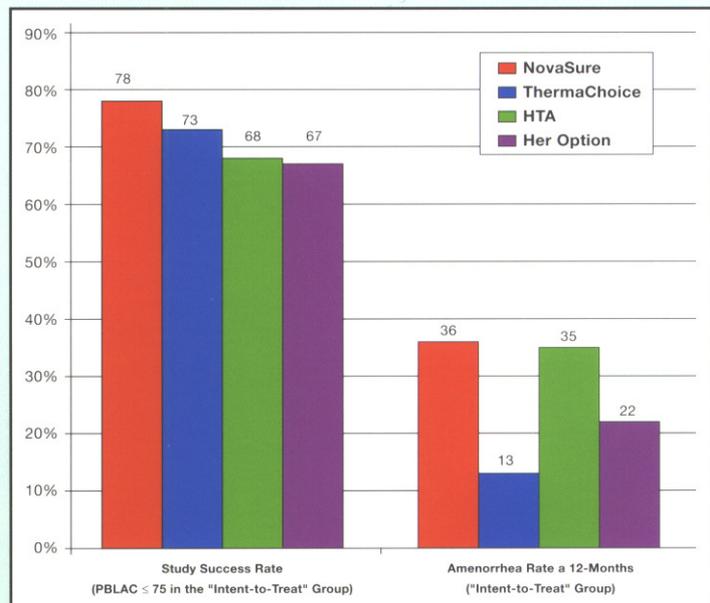
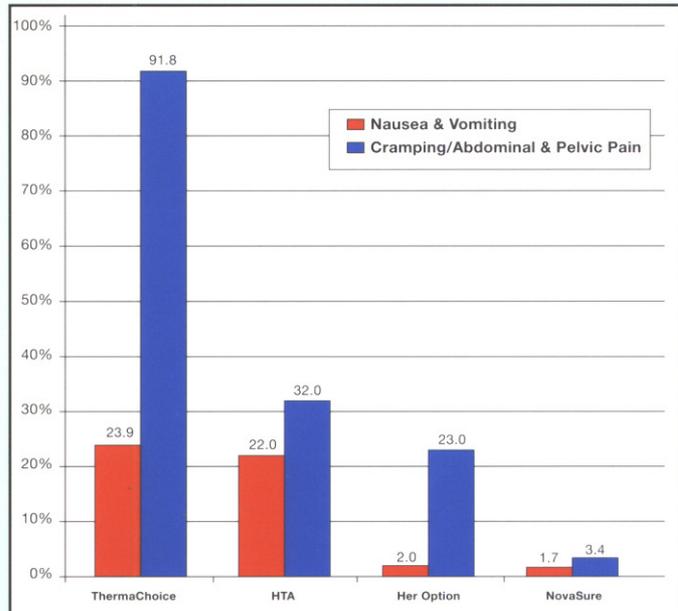
	THERMAL BALLOON (THERMACHoice)	THERMAL FLUID (HYDROTHERM-ABLATOR)	CRYOTHERAPY (HER OPTION)	IMPEDANCE-CONTROLLED (NOVASURE)
Pretreatment	Yes	Yes	Yes	No
Time of energy delivery	8 minutes	10 minutes	10-12 minutes	90-120 seconds
Cornual ablation	No	Yes	User-dependent	Yes
Principle	Ballon filled with fluid (5% dextrose in water) at 87°C	Hydrothermal circulation of saline at 90°C	Probe with transfer media creates ice ball at -100 to -120°C	Bipolar, radiofrequency ablation at 100°C
Direct visualization	None	Hysteroscopy	Ultrasound guidance	None
Safety features	Pressure and temperature-sensing cutoffs	Fluid loss detection system	Ultrasound guidance	Uterine cavity integrity assessment system

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1. Cooper, J., and Gimpelson, R. Summary of Safety and Effectiveness Data from FDA. A Valuable Source of Information on the Performance of Global Endometrial Ablation Devices. The Journal of Reproductive Medicine® Vol. 49 No. 4, April 2004.

- ThermaChoice-<http://www.fda.gov/cdrh/pdf/p970021.pdf>
- Her Option-<http://www.fda.gov/cdrh/pdf/p000032b.pdf>
- HTA-<http://www.fda.gov/cdrh/pdf/P000040b.pdf>
- NovaSure-<http://www.fda.gov/cdrh/pdf/P010013b.pdf>

2. Laberge, P., Sabbah, R., Fortin, C., Galliant, A. Assessment and Comparison of Intraoperative and Postoperative Pain Associated with NovaSure and ThermaChoice Endometrial Ablation System. The Journal of the American Association of Gynecologic Laparoscopists, May 2003, Vol. 10, No.2.

3. Brzozowski, P., and Liu, J., Surgical Techniques, OBG Management, July 2004 15-24.

**NOVASURE**

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